**speech**

# **ClearReader<sup>+</sup>**

**User manual**  
**Gebruikershandleiding**  
**Manuel d'utilisation**

**Manuale utente**  
**Manual del usuario**  
**Bedienungsanleitung**

**Optelec**

---

**Contents**

<b>1. Introduction.....</b>	<b>20</b>
1.1. About this manual .....	20
1.2. What's in the box.....	21
<b>2. Getting to know your ClearReader+ .....</b>	<b>22</b>
2.1. The ClearReader+ Family.....	22
2.1.1. ClearReader+ .....	22
2.1.2. ClearReader+ Basic.....	22
2.1.3. Magnification Feature Pack.....	22
2.2. The Optelec ClearReader+ and ClearReader+ Basic layout .....	23
2.3. Buttons.....	25
2.4. Connections .....	26
2.5. The Magnification Feature Pack buttons.....	27
<b>3. Getting Started.....</b>	<b>29</b>
3.1. Using the ClearReader+ for the first time.....	29
3.2. Opening the camera arm .....	29
3.3. Positioning a document.....	29
3.4. Turning on the ClearReader+ .....	30
3.5. Standby mode.....	30
3.6. Turning off the ClearReader+ .....	30
3.7. Reading text.....	31
3.8. Reading tables and spreadsheets.....	31
3.9. Adjusting the volume.....	31
3.10. Adjusting the reading speed.....	31

---

3.11. Play or pause reading .....	32
3.12. Spelling a word.....	32
3.13. Navigate back in the text.....	32
3.14. Navigate forward in the text .....	32
3.15. Navigate to the beginning of the document.....	32
3.16. Selecting the reading voice or language .....	33
<b>4. The Magnification Feature Pack.....</b>	<b>34</b>
4.1. Introduction .....	34
4.2. Connecting the Magnification Feature Pack to the Optelec ClearReader+ .....	34
4.3. Magnification Modes .....	34
4.4. Page Mode.....	35
4.5. Column mode.....	35
4.6. Line mode .....	35
4.7. Overview .....	35
4.8. Reading and navigating with the Magnification Feature Pack.....	36
<b>5. The ClearReader+ Menu .....</b>	<b>37</b>
5.1. Activating and closing the menu.....	38
5.2. Using the menu .....	38
5.2.1. Save .....	39
5.2.2. Open .....	42
5.2.3. Delete.....	44
5.2.4. Voice .....	44
5.2.5. View .....	46
5.2.5.1. Color .....	46

<b>Optelec</b>	<b>ClearReader+</b>
5.2.5.2. Mode .....	47
5.2.5.3. Highlight .....	47
5.2.5.5. Font .....	48
5.2.6. Settings .....	48
5.2.6.1. Voice .....	48
5.2.6.2. Options .....	49
5.2.6.2.1. Text recognition quality .....	50
5.2.6.2.2. Automatic language detection .....	50
5.2.6.2.3. Column Filter .....	51
5.2.6.3. Resolution .....	51
5.2.6.4. Restore defaults .....	51
5.2.6.5. About .....	52
<b>6. Power and battery use .....</b>	<b>53</b>
6.1. Checking the ClearReader+ battery status .....	53
6.2. Charging the ClearReader+ battery .....	53
6.3. Extending the ClearReader+ battery life .....	53
<b>7. Troubleshooting .....</b>	<b>54</b>
<b>Appendix A: Safety and Regulatory Notices .....</b>	<b>55</b>
<b>Appendix B: Certifications .....</b>	<b>57</b>
<b>Appendix C: Technical information .....</b>	<b>64</b>

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<b>1. Inleiding .....</b>	<b>67</b>
1.1. Over deze handleiding .....	67

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# 1. Introduction

Congratulations on choosing the ClearReader+, an intuitive portable reading assistant developed by Optelec.

The Optelec ClearReader+ combines ease of use with high quality multilingual speech output and fast text recognition. It can capture any printed material such as letters, newspapers and books, and reads text aloud with a naturally sounding voice. The built-in speakers provide high-quality stereo sound.

If you have any questions or suggestions concerning the use of the Optelec ClearReader+, please contact your Optelec distributor using the contact information on the last page of this manual. Your feedback is greatly appreciated and we hope you enjoy using your ClearReader+.

## 1.1. About this manual

At Optelec, we are constantly improving our products and their functionalities. Therefore, it is possible that this manual is not the latest version. Please download the most up-to-date manual on [www.optelec.com](http://www.optelec.com) at the Support section.

This manual will familiarize you with the features and easy operation of the ClearReader+. Please read this manual carefully before using your Optelec ClearReader+.

## 1.2. What's in the box

Please note that there are three types of ClearReader+. Refer to the lists below to see what should be in the packaging of the ClearReader+ you received.

The **ClearReader+** packaging contains the following:

- The Optelec ClearReader+
- A power supply
- A power cord
- This user manual

The **ClearReader+ Basic** packaging contains the following:

- The Optelec ClearReader+
- A power cord
- This user manual

The **Magnification Feature Pack** packaging contains the following:

- The Magnification Feature Pack
- A SD card

If any of these items are missing from your package, please contact your Optelec distributor.

## 2. Getting to know your ClearReader<sup>+</sup>

This chapter will familiarize you with your Optelec ClearReader<sup>+</sup>.

### 2.1. The ClearReader<sup>+</sup> Family

There are three members of the ClearReader<sup>+</sup> Family: ClearReader<sup>+</sup>, ClearReader<sup>+</sup> Basic and Magnification Feature Pack. This user manual will elaborate on the use of all ClearReader<sup>+</sup> types. Please note that not all functionalities described in this manual apply to all ClearReader<sup>+</sup> types.

#### 2.1.1. ClearReader<sup>+</sup>

The ClearReader<sup>+</sup> features an integrated battery. Thanks to this, you can enjoy reading without the need to be in close proximity of a mains power outlet.

#### 2.1.2. ClearReader<sup>+</sup> Basic

The ClearReader<sup>+</sup> Basic does not have an integrated battery, but is AC powered. To use the ClearReader<sup>+</sup> Basic, please connect to a mains power outlet to power the ClearReader<sup>+</sup> Basic.


#### 2.1.3. Magnification Feature Pack

The Magnification Feature Pack will add magnification options, as well as advanced navigation, viewing and reading functionality to your ClearReader<sup>+</sup>. Use the ClearReader<sup>+</sup> and the Feature Pack with a monitor to magnify text.

The Magnification Feature Pack can be used with the ClearReader<sup>+</sup> and ClearReader<sup>+</sup> Basic.

---

## 2.2. The Optelec ClearReader+ and ClearReader+ Basic layout

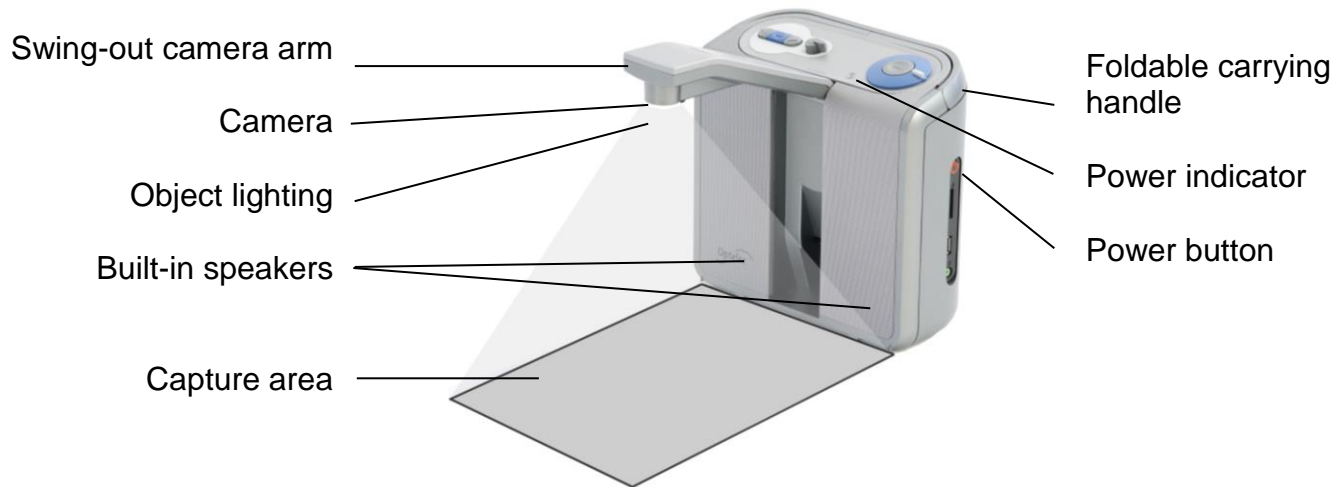Swing-out camera arm	Open the arm to capture a document.
Camera	To take a snapshot, position the document in front of the ClearReader+. Use the document positioning indicators on the bottom left and right of the ClearReader+ to position the document. The document area is approximately the width of the ClearReader+ and the length of a regular A4 or letter size page.
Object lighting	The ClearReader+ automatically provides sufficient lighting when taking a snapshot.
Built-in stereo speaker	Listen to your letters, newspaper or books.
Foldable carrying handle	Open the handle to lift and carry the ClearReader+ comfortably.
Power indicator	Light is on when the power is on, blinks in standby mode, and is off when the power is off.
Power button	Press the round button on the right-hand side to turn the ClearReader+ on or off.

## 2.3. Buttons

Play / Pause button

Speed dial

Volume dial

Back button

Forward button

Scan button

## 2.4. Connections

Power connector


Connect the power supply to your ClearReader+.

Audio out

Connect audio output device if needed.

## 2.5. The Magnification Feature Pack buttons

The following figure shows the Magnification Feature Pack buttons:

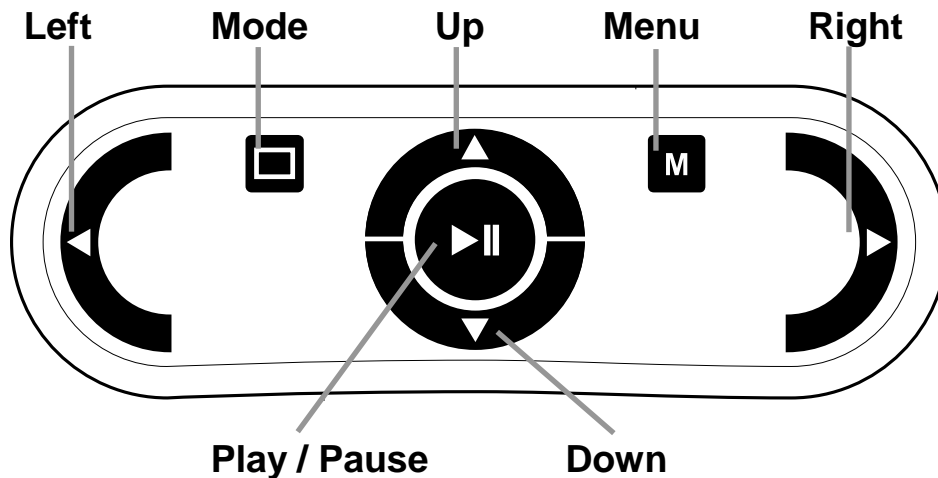**Play / Pause button:** Round button in the centre of the remote control to stop and restart reading. Press and hold down the button for 2 seconds to spell the current word. Pressing the Play / Pause button within the menu will activate the button, cycle through the available options, check or uncheck the selection or enter a submenu.

Left arrow key: Press to navigate to the previous word or to the previous menu item in the menu. During playback of the text, this button will decrease the reading speed.

---

Right arrow key: Press to navigate to the next word or to the next menu item in the menu. During playback of the text, this button will increase the reading speed.

---

Up arrow key: Press to navigate a line up in the text, or to move to the previous menu item in the menu.

---

Down arrow key: Press to navigate a line down, or to move to the next menu item in the menu.

---

Mode button: Square button on the left half of the remote control. Short press to switch between the viewing modes: Column, Line, Page high contrast, Page full color and Overview mode. Long press to hear the page orientation.

---

Menu button: Square button marked with the character “M” on the right half of the remote control. Press to enter and exit the menu.

---

## 3. Getting Started

### 3.1. Using the ClearReader+ for the first time

When you use the Optelec ClearReader+ for the first time, the battery needs to be charged. To charge the battery, connect the ClearReader+ to a mains power outlet using the ClearReader+ power supply. Charging takes approximately 3 hours.

### 3.2. Opening the camera arm

To open the camera, locate the “push” sign on the camera arm at the front of the ClearReader+ between the speakers. Gently push on the camera arm until you hear a click. The arm will eject and move towards you. Close the camera arm in the same way by pushing the end of the arm back to the closed position until you hear a click.

### 3.3. Positioning a document

The Optelec ClearReader+ camera can capture an area of an A4 or letter-size page measured from the front side of the ClearReader+. To position the document, place it in front of the ClearReader+ between the two document positioning indicators with the top edge of the document aligned towards the front of the unit. The orientation of the document will not influence the recognition of the text.

### 3.4. Turning on the ClearReader+

To turn on the ClearReader+, press the round Power button on the right-hand side. The start-up process will take approximately 40 seconds. You will hear a series of beeps to indicate that the ClearReader+ is starting up followed by the start-up sound. Now the Power indicator is lit up and the ClearReader+ is ready to use.

### 3.5. Standby mode

The ClearReader+ will go into standby mode by pressing, and holding down the Back button and Forward button for 2 seconds. The standby sound will be played, the Power indicator will start blinking and the ClearReader+ will go into standby mode. To wake up the ClearReader+ from standby, press the Power button. When the batteries are fully charged, the standby time of the ClearReader+ is approximately 40 hours. After 40 hours the ClearReader+ will power down. If switching-on the ClearReader+ from the standby mode, the start-up process will take approximately 5 seconds.

### 3.6. Turning off the ClearReader+

Press the Power button to power down the ClearReader+ completely. The power down sound will be played and the Power indicator will turn off.

### 3.7. Reading text

Press the Scan button to capture and start reading a document positioned beneath the camera. After pressing the Scan button, you will hear the snapshot sound and within a few seconds the ClearReader+ will begin reading to you.

### 3.8. Reading tables and spreadsheets

To read tables, receipts and bank statements, press and hold the Scan button for more than 2 seconds until you hear a clicking sound. A long press of the Scan button will enable the Table Reading mode and will disable the automatic column recognition, making it possible to read across columns. Text will be read from left to right ignoring column borders.

### 3.9. Adjusting the volume

Turn the Volume dial clockwise and release it to slightly increase the volume of the reading voice. Alternatively, turn the dial counterclockwise and release it to slightly decrease the volume. To rapidly adjust the volume, turn and hold the dial to the end position. When releasing the dial it will always move back to its start position.

### 3.10. Adjusting the reading speed

Turn the Speed dial clockwise and release it to slightly increase the reading speed. Alternatively, turn the dial counterclockwise and release it to slightly decrease the reading speed. To rapidly adjust the reading speed, turn and hold

---

the dial at the end position. When releasing the dial it will always move back to its start position.

### **3.11. Play or pause reading**

The ClearReader+ will automatically start reading after pressing the Scan button. To pause reading, press the Play / Pause button. Press this button once again to continue reading.

### **3.12. Spelling a word**

To spell a word, press and hold the Play / Pause button until you hear a click. Character by character, the word will be spelled out loud.

### **3.13. Navigate back in the text**

Press the Back button to navigate to the previous word. Press and hold the Back button for 2 seconds to navigate to the previous sentence.

### **3.14. Navigate forward in the text**

Press the Forward button to navigate to the next word. Press and hold the Forward button for 2 seconds to navigate to the next sentence.

### **3.15. Navigate to the beginning of the document**

To navigate to the beginning of the document, Press the Back button and Play / Pause button simultaneously.

### 3.16. Selecting the reading voice or language

To select your preferred reading voice or reading language, press and hold the Back button while turning the Speed dial at the same time. The ClearReader+ will switch to the next voice and announce the name of the newly selected voice and the language. For optimal performance the reading language and language of the document need to match. If the language of the document does not match the selected reading language, the text recognition will be of a lower quality and the recognition will take longer.

**Note:** In case automatic language detection is enabled, this button only changes between voices of the detected language. For instance, if the detected language is English this button will only change between the different English voices. With automatic language detection enabled, you will not be able to change the language of the menu. To change the menu language, either: 1) disable automatic language detection, or 2) scan a document in your preferred menu language and change the voice once in the menu.

## 4. The Magnification Feature Pack

### 4.1. Introduction

The Magnification Feature Pack will add magnification options, as well as advanced navigation, viewing and reading functionality to your ClearReader+ or ClearReader+ Basic. Use the Feature Pack without the monitor to listen to the document captured on the screen, or with the monitor to magnify it.

### 4.2. Connecting the Magnification Feature Pack to the Optelec ClearReader+

Connect the Magnification Feature Pack to the ClearReader+ and optionally, a monitor. Locate the HDMI connector at the bottom rear side of the ClearReader+. If the HDMI connector is not visible, you will need to remove the protection cover which will also reveal two USB connectors. Use a Philips screwdriver to remove the screw which closes the cover. This screw is located in the middle of the cover at the bottom side of the ClearReader+. Connect the monitor to the ClearReader+ by using a HDMI cable. Adjacent to the HDMI connector, there are two USB ports. Connect the USB cable of the ClearReader+ Magnification Feature Pack to one of the two USB ports on the rear side or to the USB port on the right side of the unit.

When the Magnification Feature Pack is connected to the ClearReader+, switch on the ClearReader+ and take a snapshot of a document as described in this manual.

### 4.3. Magnification Modes

With the Magnification Feature Pack, you can view captured documents in different viewing

---

Modes: Column, Line, Page high contrast, Page full color and overview mode. To switch between the different Modes, press the Mode button on the Magnification Feature Pack.

To set the default viewing mode and options adjust the appearance of the text such as size, color, word highlight, or font can be found within the ClearReader+ Menu.

#### **4.4. Page Mode**

The Page Mode shows a photograph of the document in its original layout, including the pictures. There are 2 Page modes, the high contrast Page mode and full color Page mode. In the high contrast Page mode the text is displayed in the selected high contrast colors.

#### **4.5. Column mode**

In the Column mode the text is reformatted and displayed in a column to fit the width of the screen regardless of the text size. Pictures are not shown in this mode.

#### **4.6. Line mode**

In Line mode the text is reformatted and displayed in one continuous line, and the highlighted word is always displayed at the centre of the screen. Pictures are not shown in this mode.

#### **4.7. Overview**

In the Full page overview, the full page is being shown with blue numbered bullets marking the detected text zones in the document. The navigation bullets can be used to quickly navigate through a text. Navigate between the bullets by pressing the Play / Pause button plus the Up

arrow to go to the previous zone and the Down arrow to go to the next zone. The speech will start reading the selected text zone. The number of the navigation bullets represent the most likely order of the text. The speech reads the zone with navigation bullet number 1, followed by navigation bullet number 2, and so on.

#### **4.8. Reading and navigating with the Magnification Feature Pack**

You can navigate to the next or previous line of the text by pressing the Up or Down arrow keys. To move a screen up in a document, press the Play / Pause button together with the Up arrow key. To move to the next screen, press the Play / Pause button together with the Down arrow key. In page overview, pressing Play / Pause together with the up or down arrow will navigate to the previous or next bullet or text zone. To go to the start or end of the document, use the Left plus Up arrow keys or Right plus Down arrow keys respectively. If you are reading a book press the Play / Pause button in conjunction with the Right arrow key to go to the next page. To go to the previous page, press the Play / Pause button with the Left arrow key.


By default the ClearReader+ will recognize columns. If you wish to disable the column recognition to read tables, press and hold the Play / Pause button and press the View button.

To start or stop reading, press the Play / Pause button. While reading, the text will scroll automatically. To decrease the reading / scrolling speed, press the Left arrow key. Alternatively, press the Right arrow key to increase the reading / scrolling speed.

A long press of the View button will announce the page orientation.

## 5. The ClearReader+ Menu

This chapter will familiarize you with the ClearReader+ Menu.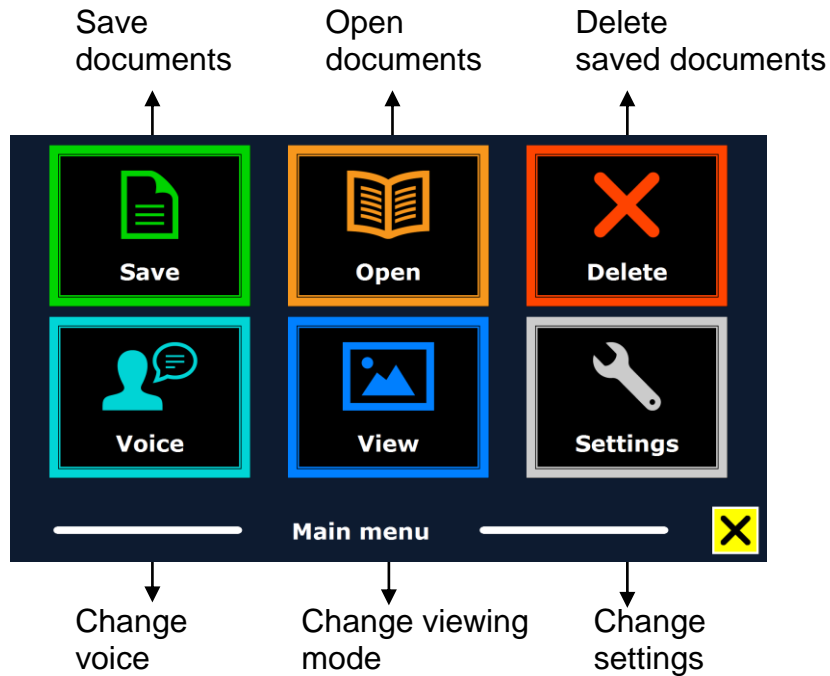

## 5.1. Activating and closing the menu

Without the Magnification Feature Pack, to activate the menu, press the Play / Pause button together with the Forward button on the ClearReader+. With the Magnification Feature Pack, press the square button on the Feature Pack marked with the character “M”.

To close the menu, press the Play / Pause button together with the Forward button again, or use the Menu button on the feature pack.

Action	ClearReader+	Magnification Feature Pack
Open Menu	Play + Forward	Menu
Close Menu	Play + Forward	Menu

## 5.2. Using the menu

The ClearReader+ menu contains two rows of three large buttons. To activate a button, navigate to the button using the Back button and Forward button on the ClearReader+. Press the Play / Pause button to activate the selected menu button. If you have a Magnification Feature Pack, use the Up, Down, Left and Right arrows to navigate through the menu. Press the Play / Pause button to select a menu item. Activating a menu button with the Play / Pause button will either change a setting, or open a submenu or list. If a button is greyed out, the option is not available.

Action	ClearReader+	Magnification Feature Pack
Next Menu Item	Forward	Arrow Right / Arrow Down
Previous Menu Item	Back	Arrow Left / Arrow UP
Select / Activate	Play	Play

The menu contains the following buttons:



- Save
- Open
- Delete
- Voice
- View
- Settings

### 5.2.1. Save

To be able to save documents, you will need to insert a SD card into the SD slot above the USB port on the right-hand side of the ClearReader+. Alternatively, you can use a USB memory stick to save documents. By default a document will be saved on the SD card. If you wish to save your document on a USB drive, connect a USB drive to the ClearReader+. When a USB drive is connected, the document will always be saved on the USB drive.

To save a document, open the menu and navigate to the blue Save button with the Back and Forward button on the ClearReader+ or the arrow keys on the Magnification Feature Pack. Activate the Save button with the Play / Pause button.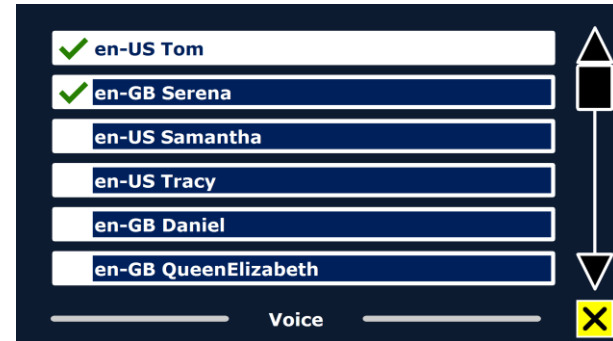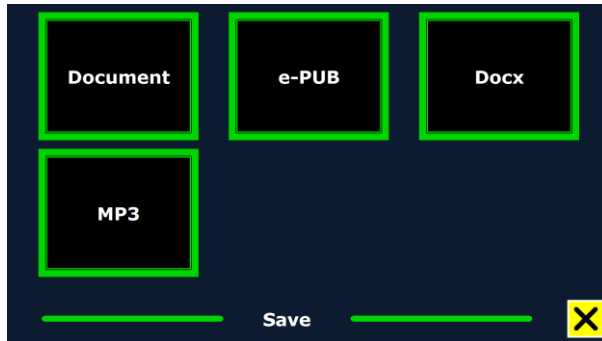

First you will be asked in what format you would like to save the document:

- Document: this is the standard ClearReader+ format
- e-PUB: This is an e-reader or Daisy compatible format
- DOCX: Use this document if you would like to edit the document on a PC with Microsoft Word
- MP3: Selecting this format will save the document as an MP3 audio file.

When saving a document as an audio MP3 file, you will need to select the reading voice. The list of voices to choose from are the Nuance voices of the selected language, or the voices of the languages detected if automatic language detection is enabled.

Not all voices can be used to save as an MP3 audio file. The available voices are limited to only the Nuance voices. In case no Nuance voice is installed, the MP3 button will be greyed out and this option is not available for the selected language.

Only one voice will be used for the entire MP3 file. Automatically switching between languages within an MP3 audio file is not supported.

Saving a document in MP3, DOCX or e-PUB takes longer than the standard format. Especially saving a multipage document in MP3 format could take several minutes.

After you selected the format to save the document in, you will be asked if you would like to save your document with a voice label.

If you wish to save a document without a voice label, select “No”. The document will be saved using the first line of the document as the name of the document. If you would like to add a voice label, select “Yes”. You will hear a beep after which you have 5 seconds to record a voice label. At the end of the recording you will hear another beep. The document will be saved with the recorded voice label.

When the document is saved, the ClearReader+ will ask if you would like to append more pages to the document.


If you select “No”, the menu will be closed, showing the document again. If you select “Yes”, you can start appending more pages to the document by pressing the Scan button. To stop appending pages to the document, press the Menu button marked with the “M” or the Play / Pause key on the ClearReader+.

The ClearReader+ will prompt you with the question if the captured pages should be processed. If “No” is selected, the conversion to text will be done while reading the document. If “Yes” is selected, all pages will be processed before returning to the document reading mode. This process may take several minutes, depending on the number of pages of the document.

### **5.2.2. Open**

To open a document, open the menu and navigate to the green Open button with the Back and Forward button on the ClearReader+ or the arrow keys on the feature pack. Activate the Open button with the Play / Pause button. Select the “Document” button to see the list of saved

documents, showing the most recently saved document first. To open and view pictures, select the “Pictures” button.
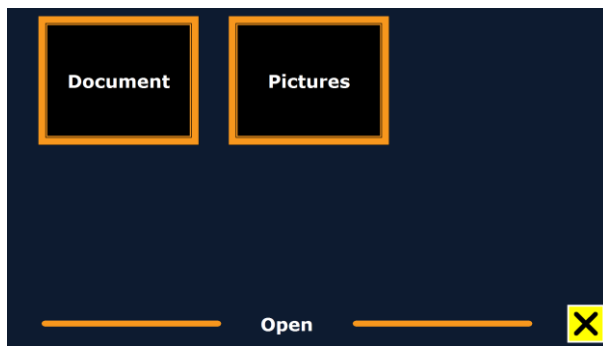


Scroll through the list of documents or pictures using the Back button or Forward button on the ClearReader+, or the arrow keys on the Magnification Feature Pack. Open a document by pressing the Play / Pause button.

In the list with documents or pictures, use the Play / Pause button to open the document or picture you wish to read or view. The menu will be closed and the document or picture will be opened. It is also possible to open text documents in “txt” format or photos from a USB stick. To open documents in “txt” format, the document should be located in the root of the USB stick or SD card.

### 5.2.3. Delete

To delete a document, open the menu and navigate to the Delete button with the Back and Forward button on the ClearReader+ or the arrow keys on the Magnification Feature Pack. Activate the Delete button with the Play / Pause button. Select the document you wish to delete.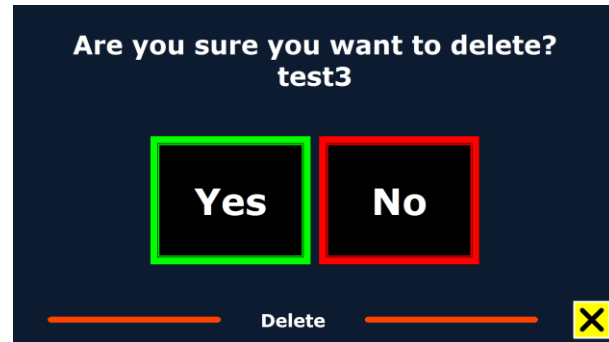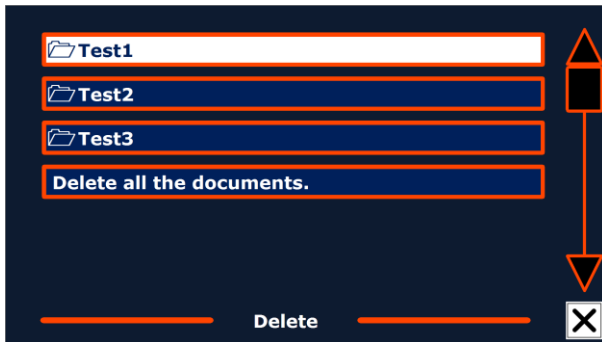


You will be prompted to confirm to delete the document. To delete the document, select “Yes”. To cancel deleting the document, select “No”. Both choices will return you to the list of documents.

### 5.2.4. Voice

This menu item lets you select your reading voice and language. To select another reading voice, open the menu and navigate to the Voice button with the Back and Forward button on the

ClearReader+ or the arrow keys on the Magnification Feature Pack. To change the voice or language, press the Play / Pause button.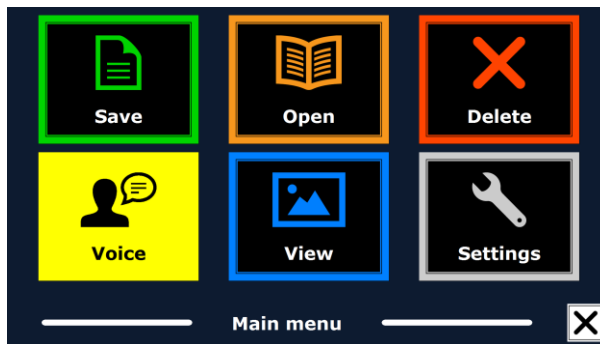


The ClearReader+ will switch to the next configured voice. If adjusting the reading voice results in a change of language, the menus will automatically change to the same language. To add or delete voices and languages from the voice selection, please refer to the Settings Menu.

**Note:** In case automatic language detection is enabled, this button only changes between voices of the detected language. For instance, if the detected language is English this button will only change between the different English voices. With automatic language detection enabled, you will not be able to change the language of the menu. To change the menu language, either: 1) disable automatic language detection, or 2) scan a document in your preferred menu language and change the voice once in the menu.

### 5.2.5. View

The “View” menu item is only available when a monitor is connected. By selecting the “View” button, the “View” menu will be opened. The “View” menu contains five buttons:

- Color
- Mode
- Highlight
- Size
- Font

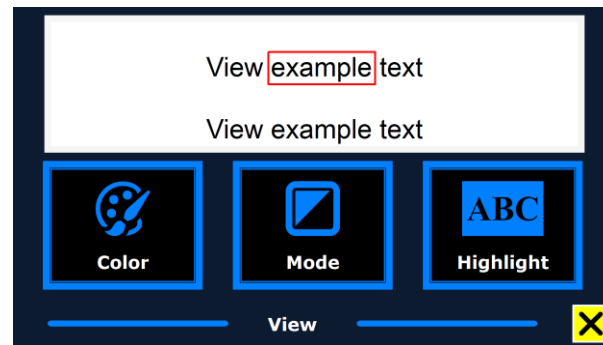#### 5.2.5.1. Color

To select another color combination, open the menu and navigate to the View button with the Back and Forward button or the arrow keys on the Magnification Feature Pack. Open the View menu with the Play / Pause button. Navigate to the Color button. Press the Play / Pause button again to cycle through the available high contrast text foreground and background color combinations. The colors that can be selected are:

- |                  |                 |                  |                 |
|------------------|-----------------|------------------|-----------------|
| • Black – White  | • Yellow – Blue | • Black – Green  | • Blue – White  |
| • White – Black  | • Blue – Yellow | • Green – Black  | • White – Blue  |
| • Yellow – Black | • Black – Red   | • Black – Purple | • Black – Amber |
| • Black – Yellow | • Red – Black   | • Purple – Black | • Amber – Black |

**Note:** This option is not available in Page full color mode.

### 5.2.5.2. Mode

To select the default viewing mode, open the menu and navigate to the View button with the Back and Forward button or the arrow keys on the Magnification Feature Pack. Open the View menu with the Play / Pause button. Navigate to the Mode button. Press the Play / Pause button again to cycle through the available different viewing modes. The viewing modes that can be selected are Column, Line, Page high contrast and Page full color.

The Page Mode shows a photograph of the document in its original layout, including the pictures. There are 2 Page modes, the high contrast Page mode and full color Page mode. In the high contrast Page mode the text is displayed in the selected high contrast colors.

In the Column mode the text is reformatted and displayed in a column to fit the width of the screen regardless of the text size. Pictures are not shown in this mode.

In Line mode the text is reformatted and displayed in one continuous line, and the highlighted word is always displayed at the centre of the screen. Pictures are not shown in this mode.

### 5.2.5.3. Highlight

This option lets you choose a word highlight, line highlight, inverted block highlight, or no highlight. Both the color of the word and line highlight can be changed from red to yellow. The use of the inverted block highlight is limited to the Column and Line view. In Page view and Overview the inverted block highlight will change into an inverted Box highlight. The red word highlight is the default.

#### 5.2.5.4. Size

This option lets you change the font size. In Column and Line view the font size is defined in points, similar to the font size in Microsoft Word. In Page view the font size is defined in steps.


#### 5.2.5.5. Font

This option lets you choose between an Arial, Arial Bold, Verdana or Verdana Bold font for the Column or Line view. In Page view the original picture and font of the document are displayed.

#### 5.2.6. Settings

Select the “Settings” button to enter the Settings menu. The Settings menu contains five buttons:


- Voice
- Options
- Resolution
- Defaults
- About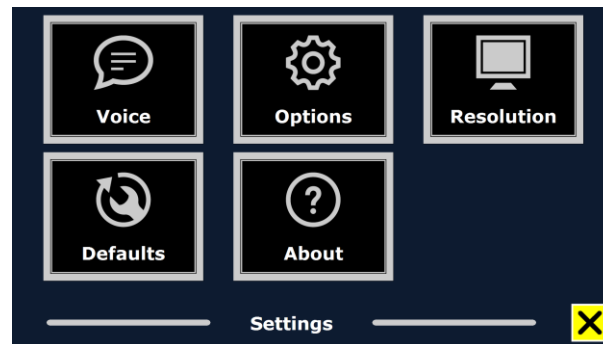

##### 5.2.6.1. Voice

The Voice button in the Settings menu opens a list with all available reading voices and languages. Selecting this button will open the languages list.

To add or remove a reading voice, select the language from the list by using the Back button or Forward button on the ClearReader+ or the arrow keys on the Magnification Feature Pack.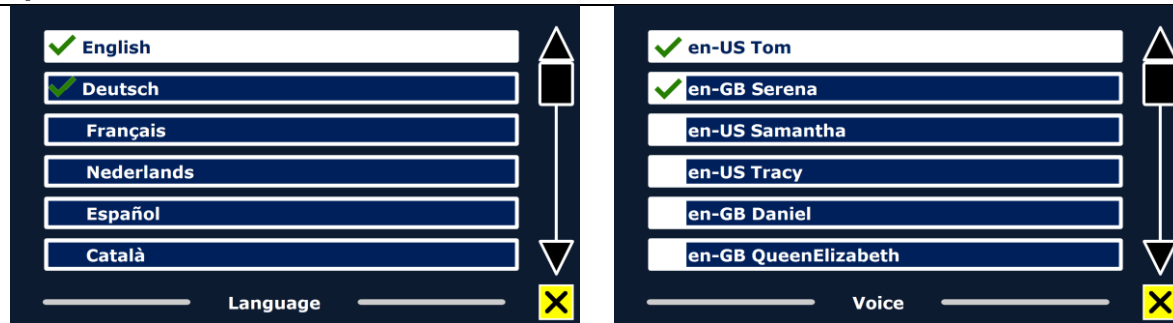


Press the Play button to select the language and to view the list with the available voices for that language. To select or deselect a voice, press the Play / Pause button. A checkmark in front of the voice name will indicate if it is enabled. When the voice is enabled, it will be available under the “Voice” button in the main menu. By deselecting a voice, the voice will be removed from the available reading voices. The languages of which a reading voice is selected, will appear at the top of the list with languages, and be marked with a checkmark.

### 5.2.6.2. Options


The options menu contains the following configuration settings:

- Text recognition quality
- Automatic language detection
- Column filter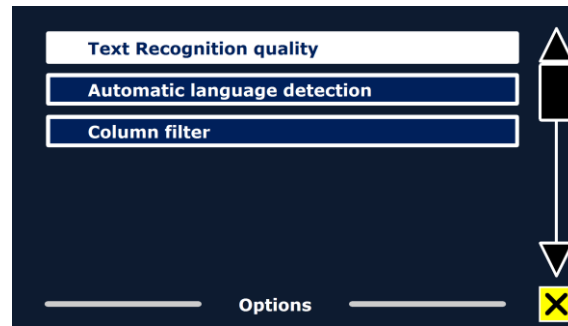


### 5.2.6.2.1. Text recognition quality

This option lets you set the recognition quality. The “Optimal” setting is the recommended mode and will offer the best text recognition quality. The “Fast” mode is for simple documents without columns and pictures. The “Fast” mode is up to 30% faster than the “Optimal” mode.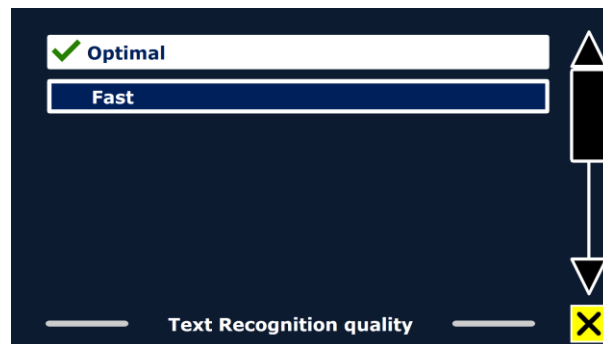

### 5.2.6.2.2. Automatic language detection

When the “Automatic language detection” is enabled, the ClearReader+ will detect the language of the text reading zone or paragraph, and automatically select the correct reading voice. The ClearReader+ is capable of detecting multiple languages in one page. To enable the language detection, first enable up to maximum five languages. To do this, go to the “Voice” selection menu, and enable a language by selecting at least one voice per language. Do this for a maximum of 5 languages. Now make sure the language detection is enabled in the “Options” menu.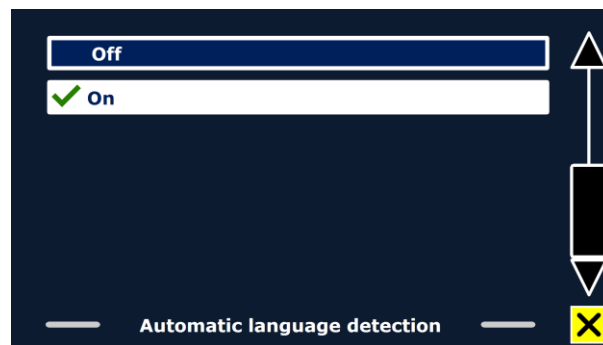

**Note:** In case the ClearReader+ is not able to detect the language, the language of the menu is being used. If English is one of the selected languages, the ClearReader+ will use an English voice and will ignore the menu language.


### 5.2.6.2.3. Column Filter

This option contains two settings:

- Ignore incomplete columns
- Include incomplete columns

When “Ignore incomplete columns” is enabled, the OCR algorithms will try to detect if text columns are cut off at the left or right margin of the picture.

Disabling the filter will include partial columns and will read all detected text on the page. The “Ignore incomplete columns” is the recommended and default setting, offering the most comprehensive text recognition. The “Include incomplete columns” will try to detect all text on the page, but may result in incomprehensive text at the beginning or end of the document.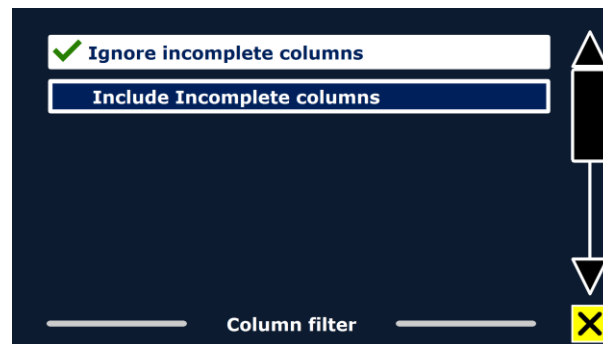


### 5.2.6.3. Resolution

This option lets you manually set the monitor resolution. It is recommended to only change the resolution if the automatic detection fails. After changing the resolution, the ClearReader+ needs to restart.

### 5.2.6.4. Restore defaults

If you wish to restore the default settings, select this button. You will be prompted if you are sure you want to restore the defaults. To restore the defaults, select “Yes”. To cancel restoring the defaults, select “No”.

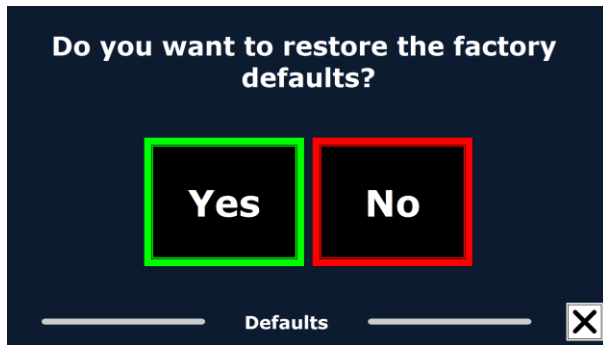### Default settings

<b>View</b>	High contrast on
<b>Color</b>	Black on white
<b>Highlight</b>	Word

#### 5.2.6.5. About

The About screen shows the version of the Application interface, camera interface, OCR interface and the mains frequency.

## 6. Power and battery use

This chapter only applies to the ClearReader+. The ClearReader+ is the only ClearReader that has an integrated battery. The ClearReader+ Basic is AC powered and needs to be powered by a mains power source.

### 6.1. Checking the ClearReader+ battery status

Press the Back and Forward buttons together to check the battery status of the ClearReader+. The ClearReader+ will announce the status in percentage, where 100% means a fully charged battery. If the battery is low, please charge the battery with the dedicated power supply.

### 6.2. Charging the ClearReader+ battery

To charge the ClearReader+, connect the power supply to the power connector located at the rear side of the unit. Plug the power cord into a power outlet.

Charging the ClearReader+ takes approximately 3 hours. A fully charged ClearReader+ can operate for approximately 5 hours. Please note that, for safety reasons, only the power supply supplied by Optelec should be used to charge the ClearReader+.

### 6.3. Extending the ClearReader+ battery life

To ensure the battery pack maintains maximum capacity, disconnect the ClearReader+ from the power source at least once a month and operate it on battery power until the battery pack is empty.

---

## 7. Troubleshooting

---

If the power supply does not power the ClearReader+.

- Check the connections. Make sure that the power supply is firmly connected to the ClearReader+ and to the power outlet.
  - If the power supply still does not power the unit, contact your Optelec distributor.
- 

If the battery does not power the ClearReader+.

- The battery may be empty. Connect the power supply to charge the battery.
- 

If the text you are reading does not make sense.

- Make sure your reading language is the language of your document.
- 

If the text recognition takes unusually long before starting to read your document.

- Make sure your reading language corresponds with the language of your document.
- 

If the ClearReader+ does not read the whole line of a document / column / table.

- The automatic column recognition may be on. Press and hold the Scan button for 2 seconds to disable the column recognition and to switch to “TableMode”.
- 

If the ClearReader+ does not respond to the Scan button.

- Open the camera arm and try it again.
  - If the Scan button still does not respond, check if the unit is powered.
- 

If the ClearReader+ does not respond to any button.

- Make sure the battery is not empty. If the battery is empty, connect the power supply to charge the battery.
-

---

## Appendix A: Safety and Regulatory Notices

- Handle the ClearReader+ with care. Rough handling will damage internal components.
- Do not service the ClearReader+ yourself. For service or questions on how to use the ClearReader+, please contact your Optelec distributor.
- **Do not open or remove any parts** of the ClearReader+ as it will void the warranty.
- Do not try to remove the battery.
- Do not expose the ClearReader+ to excessive heat or direct sunlight to avoid the risk of fire.
- Do not use the ClearReader+ near inadequately shielded medical devices.
- Do not use the ClearReader+ for medical purposes. The ClearReader+ should not be part of, be used for, or be used during a medical treatment or diagnosis.
- To avoid risk of electrical damage, keep your ClearReader+ away from fluids and chemicals.
- Always unplug the ClearReader+ before cleaning. Use a soft cloth to clean the exterior. Do not use cleaning agents or abrasive materials as they will damage your unit.
- All materials used to build the ClearReader+ are thoroughly tested and meet the required safety regulations. However, this is no warranty against potential allergic reactions.
- For safety reasons, please place the ClearReader+ on a stable surface and keep the ClearReader+ away from pets.
- Only use a power cord that matches the voltage of the power outlet that is approved and complies with the safety standard of your country.
- The ClearReader+ is designed for domestic use, use in the office and use for education purposes.
- **Caution:** No modification of the power supply is allowed.

- **Make sure any input or output connection is properly connected using the appropriate cabling and connectors, and that the input is not connected to the output.**
- **Caution:** Do not replace the battery pack by an incorrect type. Please dispose the used battery pack according to the instructions.
- In case the battery does not charge or if there is any other malfunctioning, please power off the ClearReader+, unplug the device from the mains power and contact your Optelec distributor for assistance.

### Please take notice of the information on the product label:

	<p>Follow the operating instructions when using the ClearReader+</p>		<p>Caution</p>
	<p>The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste.</p>		<p>Keep dry</p>
	<p>Stand by</p>		

## Appendix B: Certifications

### WEEE Notice

The directive on Waste Electrical and Electronic Equipment (WEEE), which entered into force as European law on 3<sup>rd</sup> December 2008, resulted in a major change in the treatment of electrical equipment at end-of-life.

The purpose of this Directive is, as a first priority, the prevention of WEEE, and in addition, to promote the reuse, recycling and other forms of recovery of such wastes so as to reduce disposal.

The WEEE logo (shown at the left) on the product or its box indicates that this product must not be disposed of or dumped with your other household waste. You are liable to dispose of all your electronic or electrical waste equipment by relocating over to the specified collection point for recycling of such hazardous waste. Isolated collection and proper recovery of your electronic and electrical waste equipment at the time of disposal will allow us to help conserve natural resources. Moreover, proper recycling of the electronic and electrical waste equipment will ensure safety of human health and the environment. For more information about electronic and electrical waste equipment disposal, recovery, and collection points, please contact your local city centre, household waste disposal service, shop from where you purchased the equipment, or manufacturer of the equipment.

### RoHS Compliance

This product is in compliance with Directive 2011/65/EC of the European Parliament and of the Council of 1<sup>st</sup> July 2011, on the restriction of the use of certain hazardous substances in electrical and electronic equipment (RoHS) and its amendments.

### European Union Notice

Products with CE marking comply with the R&TTE Directive (99/5/EC), the EMC Directive (2014/30/EU), and the Low Voltage Directive (2014/35/EU) issued by the Commission of the European Community.

Compliance with these directives implies conformity to the following international standards:

IEC 60950-1;

EN 55032

EN 55024

FCC

FCC part 15 Class B

This device bears the CE and FCC approval mark

## EMC tables

### Guidance and manufacturer's declaration – electromagnetic emissions

The ClearReader+ is intended for use in the electromagnetic environment specified below. When using the ClearReader+, make sure the ClearReader+ is being used in the environment described below.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11		The ClearReader+ uses RF energy only for its internal functioning. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11		The ClearReader+ is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2		
Voltage fluctuations/ flicker emissions IEC 61000-3-3		

## Recommended separation distances between portable and mobile RF communications equipment and the ClearReader+

The ClearReader+ is intended for use in the electromagnetic environment specified below. When using the ClearReader+, make sure the ClearReader+ is being used in the environment described below.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**Guidance and manufacturer's declaration – electromagnetic immunity**

The ClearReader+ is intended for use in the electromagnetic environment specified below. When using the ClearReader+, make sure the ClearReader+ is being used in the environment described below.

<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact  ±8 kV air	±6 kV contact  ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst  IEC 61000-4-4	±2 kV for power supply lines  ±1 kV for input/output lines	±2 kV for power supply lines  ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s)  ±2 kV line(s) to earth	±1 kV line(s) to line(s)  ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
interruptions and	<5 % <i>UT</i>	<5 % <i>UT</i>	Mains power quality should

<p>voltage variations on power supply input lines</p> <p>IEC 61000-4-11</p>	<p>(&gt;95 % dip in <i>UT</i>) for 0,5 cycle</p> <p>40 % <i>UT</i> (60 % dip in <i>UT</i>) for 5 cycles</p> <p>70 % <i>UT</i> (30 % dip in <i>UT</i>) for 25 cycles</p> <p>&lt;5 % <i>UT</i> (&gt;95 % dip in <i>UT</i>) for 5 sec</p>	<p>(&gt;95 % dip in <i>UT</i>) for 0,5 cycle</p> <p>40 % <i>UT</i> (60 % dip in <i>UT</i>) for 5 cycles</p> <p>70 % <i>UT</i> (30 % dip in <i>UT</i>) for 25 cycles</p> <p>&lt;5 % <i>UT</i> (&gt;95 % dip in <i>UT</i>) for 5 sec</p>	<p>be that of a typical commercial or hospital environment. If the user of the ClearReader+ requires continued operation during power mains interruptions, it is recommended that the ClearReader+ be powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50/60 Hz) magnetic field</p> <p>IEC 61000-4-8</p>	<p>3 A/m</p>	<p>3 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE: <i>UT</i> = AC mains voltage prior to the application of the test level.</p>			

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>Vrms</p> <p>V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the ClearReader+, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b></p> <p><math>d = 1,2\sqrt{P}</math></p> <p><math>d = 1,2\sqrt{P}</math> 80 MHz to 800 MHz</p> <p><math>d = 2,3\sqrt{P}</math> 800 MHz to 2,5 GHz</p> <p>where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m).</p>

			<p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			<p>A) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ClearReader+ is used exceeds the applicable RF compliance level, please verify proper operation of the ClearReader+. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the ClearReader+.</p> <p>B) For the frequency range 150 kHz to 80 MHz, field strengths should be less than V/m.</p>

## Appendix C: Technical information

Dimensions (W x H x D):	24 x 23 x 10 cm (9.4 x 9.1 x 3.9 inches)
– Depth, camera arm open:	27 cm (10.6 inches)
– Height, camera arm open:	25 cm (9.8 inches)
– Height, handle open:	27 cm (10.6 inches)
Weight:	2.46 kg (5.4 lbs)
Camera:	High resolution camera
Earphone connection:	3.5 mm stereo jack
Battery type:	Li-ion
Charging time:	3 hours
Battery life:	Approximately 5 hours continuous use
Battery pack lifetime:	1 year at least

### Operation conditions:

Relative humidity:	<70%, no condensation
Temperature:	0 to 40 °C
Operation Pressure	700 – 1060 hPa
Operation Lifetime:	2 years at least

### Storage and transport conditions:

Storage Pressure	500 – 1060 hPa
Transportation Pressure	500 – 1060 hPa

Relative humidity: <95%, no condensation

Temperature: -20 °C to 60 °C

**Power supply**

The EDACPOWER ELECTRONICS CO.,LTD type EM10682C(01) power supply is part of the ClearReader+ medical certified device

AC/DC power supply: Co., Ltd. EDACPOWER ELECTRONICS CO.,LTD type EM10682C(01)

Input: AC 100-240V, 50/60Hz, 2.0A; Output: 20Vdc. 2.0A

Classification:

- Input rating for ClearReader+: 20Vdc. 2A
- Protection against electric shock: Class II
- Protection against harmful ingress of water or particulate matter: IPX0

This device bears the CE and FCC approval mark

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All rights reserved
U.S.A.
### Optelec GmbH